

Independent Sub-pixel Drive Technology

Whitepaper

Independent Sub-pixel Drive technology*¹ achieves advanced image quality while minimizing image degradation. This whitepaper describes the features of Independent Sub-pixel Drive technology, how it works, and the background to its development.

The Need to Enhance Display Resolution

Today’s medical imaging devices deliver excellent resolution, with a pixel size range of approximately 25 to 100 μm and a pixel count from 4 to 67 megapixels (MP). Unfortunately, medical displays have not kept pace with this progress—the pixel pitch of commonly used high-resolution 5MP displays is 165 μm, but their resolution properties cannot match those of current medical imaging devices.

What happens to an image captured by a high-resolution medical imaging device when shown on a lower-resolution display? Quality and/or image size are reduced. Either the entire native image is converted to a lower-resolution image via sub-sampling done by viewer software, or just one small portion of the native image is displayed with the original resolution (see Figure 1). When the original full-screen image is viewed on a display that supports fewer pixels, the image reduces to less than a third (31%) of the original.

Resolution of major mammography systems and a 5MP display

Mammo system	Pixel size (μm)	Resolution (pixel)
A	25	7080 x 9480 (67.12MP)
B	50	3540 x 4740 (16.77MP)
C	70	2560 x 3328 (8.52MP)
D	100	1914 x 2294 (4.39MP)
5MP Display	165	2048 x 2560 (5.24MP)

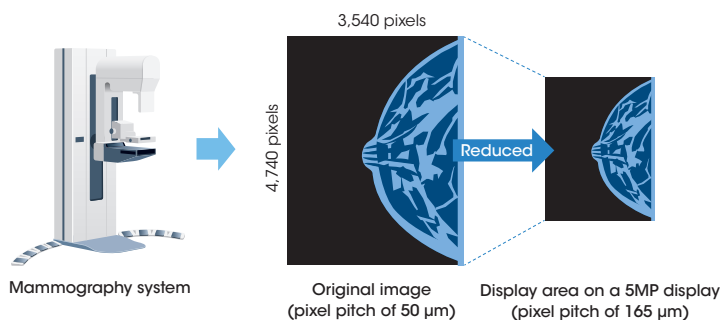


Fig. 1 Reduction of the information volume on display

Benefits of Independent Sub-pixel Drive Technology

- The display with Independent Sub-pixel Drive Technology delivers images that are faithful to the original, but with less image degradation.
- Compared with a conventional 5MP/3MP LCD, Independent Sub-pixel Drive Technology significantly improves image quality without changing the pixel structure and without any adverse impact on luminance and contrast characteristics.
- The pixel pitch of one-third in a horizontal direction (the sub-pixel chain direction) improves MTF*² and allows images to be more accurately depicted.
- Recently the luminance, contrast, viewing angle, and grayscale features of medical displays have been improved. Now the resolution of medical displays can also be improved, thanks to this cutting-edge technology.

*1 Applicable to the LMD-DM50 and LMD-DM30 only.

To use the Independent Sub-pixel Drive function, special viewer software is required. For details, please contact your nearest Sony office or authorized dealer. Independent sub-pixel drive is proprietary technology of JVC KENWOOD Corporation.

*2 MTF is the magnitude component of the true measure of resolution (image sharpness) that an imaging system can achieve.

Independent Sub-pixel Drive Technology

A conventional LCD display reproduces images on-screen by means of many small pixels. For example, a 5MP diagnostic display drives 2,560 x 2,048 pixels, while a 3MP diagnostic display drives 2,048 x 1,536 pixels. Generally the shapes of the pixels are square, and one pixel consists of three sub-pixels. (Figure 2) Sony's LMD-DM50 and LMD-DM30 Medical Displays feature Independent Sub-pixel Drive technology that enables sub-pixels to be driven independently on different luminance levels, whereas conventional LCD displays drive three sub-pixels together as only one square pixel with one luminance level.

As sub-pixels are lined up in scanning direction (the sub-pixel chain direction), the process of driving sub-pixels independently results in three times the resolution.

Because Independent Sub-pixel Drive technology does not manipulate original image data during the signal process, it achieves higher resolution on-screen by minimizing the image degradation.



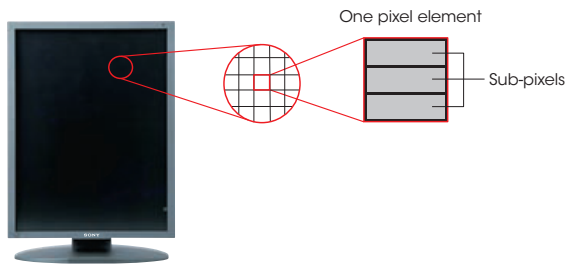


Fig. 2 Pixels and sub-pixels

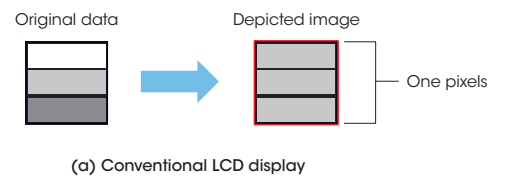

(a) Conventional LCD display

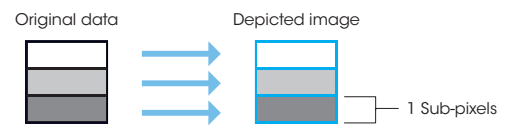


(b) LCD display with Independent Sub-pixel Drive technology

Fig. 3 Difference in luminance level

The working image of the Independent Sub-pixel Drive technology is illustrated in Figure 4. This image depicts the original with greater clarity by driving sub-pixels independently.


Figure 5 shows another comparison of the same captured image on a conventional 5 Mega Pixel (5MP) LCD display (on the left) and a display with the addition of Independent Sub-pixel Drive technology (on the right). The excellent clarity and high resolution of the Independent Sub-pixel Drive technology display is evident in the micro-calcifications, especially at the edges.

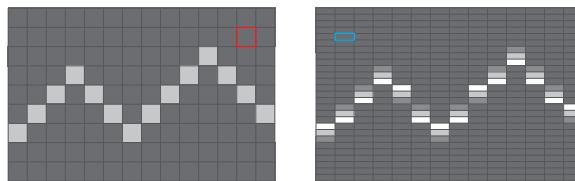

Conventional LCD

LCD with Independent Sub-pixel Drive technology

Fig. 4 Working Image of Independent Sub-pixel Drive technology

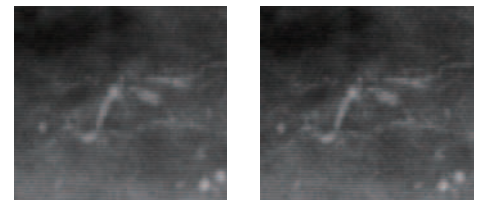

Conventional 5MP LCD display

LMD-DM50

Fig. 5 Image comparison

Distributed by

©2013 Sony Corporation. All rights reserved.
 Reproduction in whole or in part without written permission is prohibited.
 Features and specifications are subject to change without notice.
 "SONY" is trademark of Sony Corporation.
 To enhance explanation and understanding,
 images in this brochure are illustrated or simulated.
 Independent sub-pixel drive is proprietary technology of
 JVC KENWOOD Corporation.